

## *BLS Systems Introduces the* FLO MAX Filtered Oxygen Mask



### *The Next Generation in Oxygen Therapy*

- **Deliver from 30% - 99% Oxygen; Low - Medium - High oxygen concentrations with one mask.**
- **Compatible with standard Nebulizer for Drug therapy.**
- **All patient exhalation is filtered through submicron, hydrophobic 3M™ filter. Protects healthcare workers *and* patients.**

#### *Other features include*

- ❑ A closed mask breathing system with integral 3M™ filter maintains respiratory isolation during oxygen and drug aerosol therapy, reducing the risk of airborne infections such as SARS and influenza.
- ❑ Over 99.9% bacterial and viral filtration efficiency.
- ❑ Extra large oxygen reservoir ensures highest possible oxygen concentration while intake valve allows patient to make-up air from atmosphere if necessary.
- ❑ 3 mask sizes (Large Adult, Small Adult and Child) ensure best fit possible.
- ❑ Cost effective; Replace Low, Med, High Oxygen concentration and nebulizer masks with one mask.
- ❑ 100% latex free and disposable.
- ❑ Patent pending.



## The FLO<sub>2</sub>MAX Filtered Oxygen Mask

### How it works

Soft plastic and flexible aluminum nose piece help ensure airtight fit

Latex free closed system mask is made of soft durometer plastic and comes in 3 sizes to ensure comfort and fit

Intake valve allows patient to breathe from atmosphere only after oxygen reservoir is depleted

All patient exhalation passes through a 3M filter and out the exhalation valve

Ring shaped submicron, hydrophobic 3M filter provides over 99.9% BFE and VFE

Oxygen reservoir bag and inlet can be removed to accommodate a standard nebulizer

Product Number	Description
6100	Large Adult FLO <sub>2</sub> MAX Mask with O <sub>2</sub> reservoir and tubing
6100	Child FLO <sub>2</sub> MAX Mask with O <sub>2</sub> reservoir and tubing
6105	Small Adult FLO <sub>2</sub> MAX Mask with O <sub>2</sub> reservoir and tubing
6250-N	Rescuer Nebulizer

BLS Systems Limited  
1124 South Service Road West  
Oakville, Ontario  
Canada L6L 5T7

Telephone: (905)-339-1069  
Fax: (905)-339-1060  
E-mail: sales@blssystemsltd.com  
Website: <http://www.blssystemsltd.com>